

Made in JAPAN

Healthcare professional friendly design

Incident Barrier Medical Shield

I·B·M Shield

I.B. Medical Shield



Shield does not obstruct your view – distortion free and optically clear.
Easily adjustable frame with back strap.




Features

- ☆Shield covers the area around the ears to protect the sides from splashes.
- ☆Offers clear visibility without distortion.
- ☆Anti-static treatment prevents dust from easily attaching to shield.
- ☆Enough space secured even with glasses and mask on.
- ☆Size-adjustable frame and strap, from general size S to LL.
- ☆ Light with a total weight of 50g.

Instruction Manual

Assembly Part

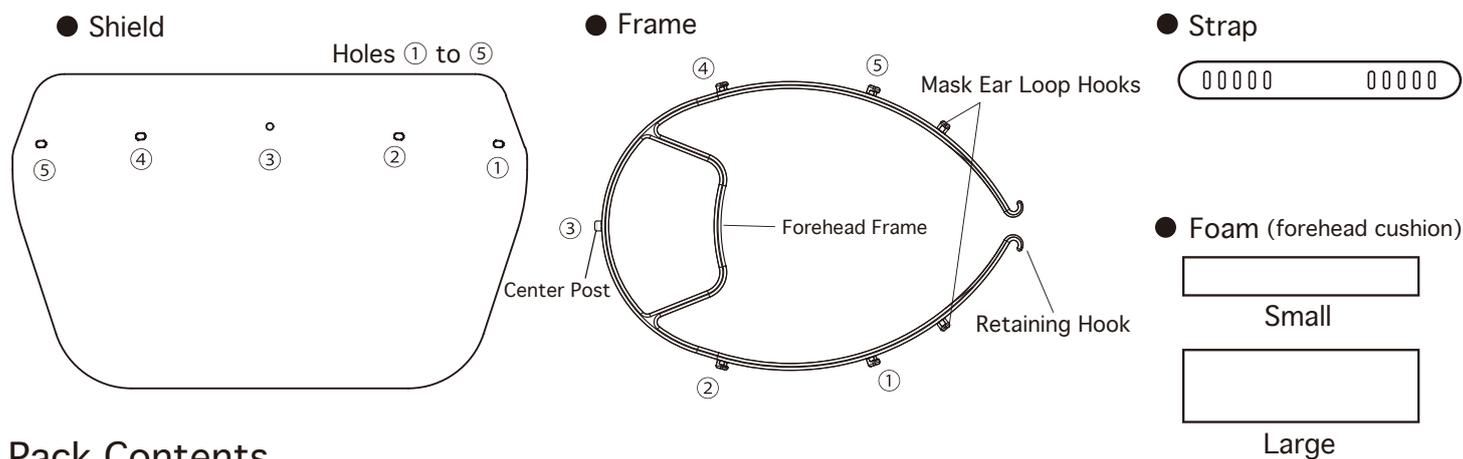


Pack Contents

Name	Size in mm	Materials	No. per pack
Shield	300×240×0.2	Polyvinyl chloride (PVC), antistatic agent	5
Frame	215×146×8	Polyacetal (POM / N / BK)	5
Strap	220×20×3	Polyolefin elastomer	5
Foam (L & S)	50X8X3 (S) 50X20X3(L)	EPT sponge	5 each


* Shields can be purchased separately.

Assembly Directions


1. Attach one side of shield holes by pushing over side frame posts ① and ②. (This makes it easier to position the shield over the center post and fit it in hole ③)

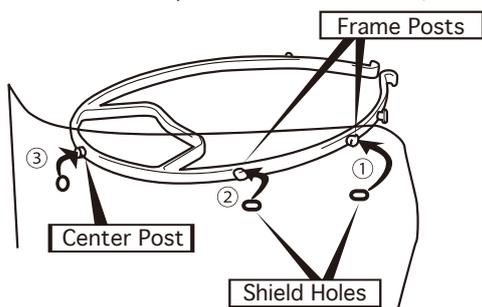

2. While holding the frame and center post with 2 fingers, attach the other side of shield holes by pushing over side posts ④ and ⑤. (Pulling the edge of the shield a little while attaching makes it easier) Ensure that all 5 posts of frame are attached securely to the shield.

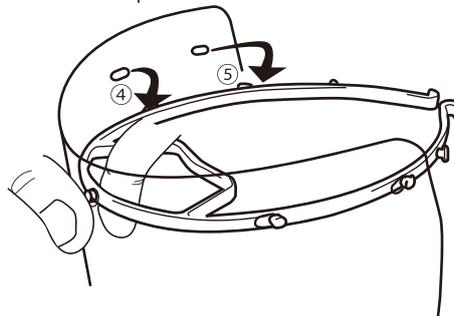


3. Hooks are provided on the side of the frame for mask ear loops. Please use them if needed.

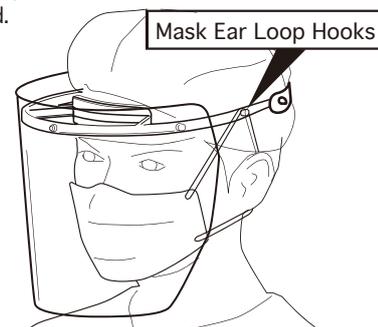


◆ Apply self adhesive foam to forehead frame

There are two types of foam, large and small. Please choose the one that suits you.

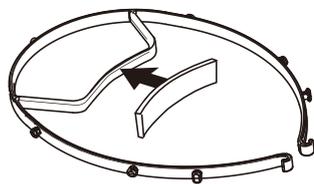


◆ Attach the strap

Insert the retaining hooks in the strap slits. There are 5 slits on each side for adjustment.

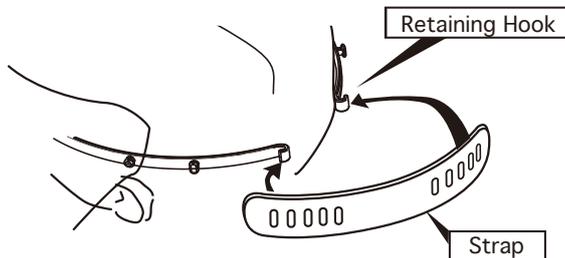

◆ Wear it with other equipment.

You can wear it with glasses on.

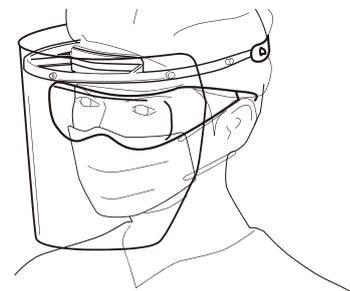

Precautions

- 1) The frame and shield are disposable.
- 2) If the shield becomes scratched or smudged, it may cause accidents from poor visibility. Replace it with a new one accordingly.
- 3) This product aims to protect the face as much as possible from fine dust and droplet dispersion. Please be aware that it does not protect the wearer's eyes and body from damage.
- 4) Do not attempt to disassemble or modify.
- 5) Take care when handling shield as edges all around are thin and may cause scratches to the skin.
- 6) To undo the strap, hold one end as you detach the other end from the frame. Then hold the other hook and strap as you detach.
- 7) When replacing the shield, please use the same genuine product.

Distributor



Medi Beat Inc.

86 Nishi-cho, Yonago City, Tottori, 683-8503, Japan
 Phone: +81-859-36-6161 Facsimile: +81-859-36-6162
 E-mail: info@medibeat-inc.co.jp
 URL: https://medibeat-inc.co.jp/