



Infant and child arm support immobilizers




# Super Mamorunja!!

Provides support for infants and young children by temporarily immobilizing both arms after surgeries such as cleft lip and palate repair surgeries.


Product created from on-field insights



Provides the same gentle support of a mother's hand.

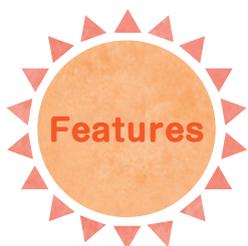


- ★ Immobilizes elbow from bending (child can still move fingers and freely hold objects)
- ★ Size freely adjustable
- ★ Made of paper and easily disposed as burnable garbage

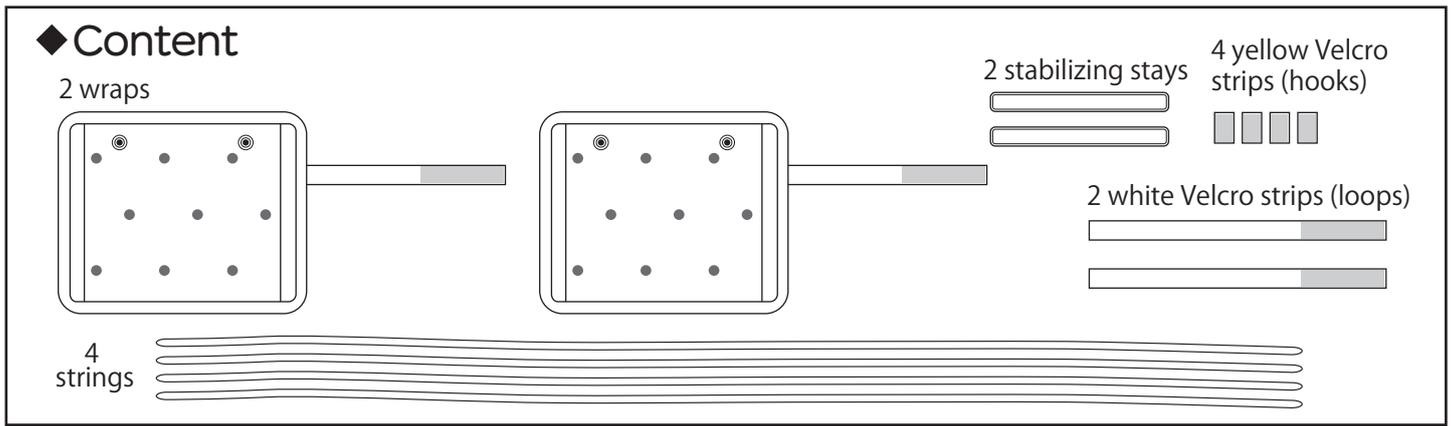


**1** Cut the Super Mamorunja wrap and adjust it to the length of the forearm. (Cut also the stabilizing stay so that it is within the length of the wrap).

\* Careful not to cut the wrong side.

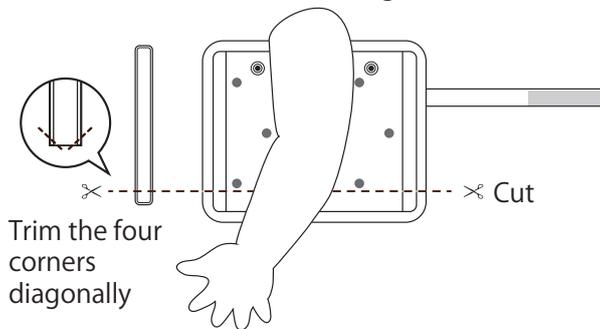


**2** Turn over the wrap and attach the white Velcro loop strip and the stabilizing stay to the cut side of the cardboard wrap.

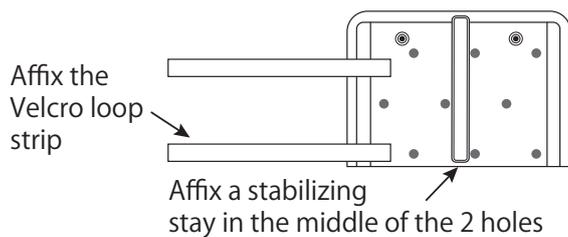


**3** On the other end of the wrap, attach the yellow Velcro hook strips.

\* Two Velcro hook strips should be attached on the same end. Please affix appropriately.

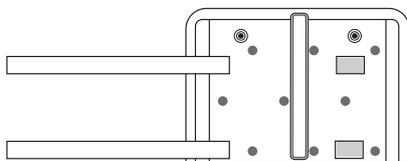


**4** Attach the supplied strings to the wrap. There are two holes for tying the strings to the wrap. Ensure they are securely fastened.

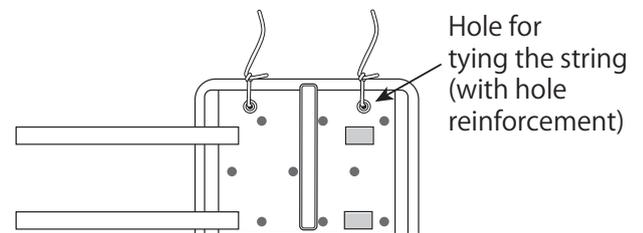

**5** Secure the wrap around the forearm with Velcro.

\* Please note the mounting position.

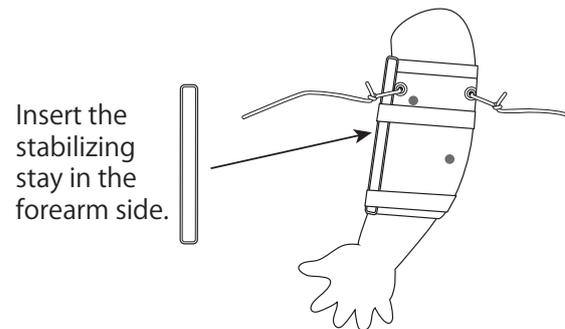

**6** Tie the strings around the back side of the neck, adjusting the length.

